

Klarity Optek™ Overlay Indexing Bar

INSTRUCTIONS FOR USE

GENERAL INFORMATION

Klarity Indexing Bars are used for patient positioning during radiation therapy and other disease treatments.

PRODUCT DESCRIPTION

Klarity Indexing Bars are used for positioning and re-positioning patient's head and neck during radiation therapy and other disease treatments.

Klarity Optek™ Overlay Indexing Bars are compatible with different couch tops to secure the Optek™ Overlay to the couch top. Please consult our catalog or www.klaritymedical.com for a complete selection.

MAINTENANCE AND STORAGE

Klarity Indexing Bars must be stored in a dry, clean place between 50°F(10°C) and 122°F(40°C). Keep away from extreme heat.

Store in a safe place to prevent damage to product.

Klarity Indexing Bars can be cleaned and disinfected with soapy water or a neutral water-based antiseptic cleaner. Allow to dry completely before next use. Do not use acidic or alkali disinfectant, or abrasive detergents.

Hinges and corresponding screws should be replaced every 5 years. Check periodically to ensure product is in good working condition.

WARRANTY

1 Year

INSTRUCTIONS FOR USE

1. Select the appropriate indexing bar for the CT Couch Top.
2. Position the indexing bar on the treatment table.
3. Place the Klarity Optek™ Overlay on top of the indexing bar.
4. Rotate the screws to securely fit the clamp against the Overlay.
5. When finished, unscrew the clamps to release the Overlay.

To be used by a qualified medical professional
IFU-A-015



Keep in a cool and dry environment



35°C
Store below 35°C (95°F)



ITEM INFORMATION

ITEM ID	PRODUCT DESCRIPTION	COMPATIBILITY
RE-630EA	Insert-Lock Aluminum Indexing Bar for Klarity Optek™ Overlay	iBEAM®, Novalis, and TomoTherapy® Couch Tops
RE-7E	Turn-Lock Aluminum Indexing Bar for Klarity Optek™ Overlay	Varian Couch Tops



Lotus NL B.V.
Koningin Julianaplein 10, le Verd, 2595AA
The Hague, Netherlands
T: +31645171879 (English)
T: +31626669008 (Dutch)

Klarity Medical Products, LLC
600 Industrial Parkway
Heath, OH 43056
T: 740.788.8107 F: 740-788-8109
info@klaritymedical.com
www.klaritymedical.com