

Klarity Indexing Bar

INSTRUCTIONS FOR USE

GENERAL INFORMATION

Klarity Indexing Bars are used for patient positioning during radiation therapy and other disease treatments.

PRODUCT DESCRIPTION

Klarity Indexing Bars are used for positioning and re-positioning patient's head and neck during radiation therapy and other disease treatments.

Klarity Indexing Bars are available for different baseplate compatibilities. Please consult our catalog or www.klaritymedical.com for a complete selection.

MAINTENANCE AND STORAGE

Klarity Indexing Bars must be stored in a dry, clean place between 50°F(10°C) and 122°F(40°C). Keep away from extreme heat.

Store in a safe place to prevent damage to product.

Klarity Indexing Bars can be cleaned and disinfected with soapy water or a neutral water-based antiseptic cleaner. Allow to dry completely before next use. Do not use acidic or alkali disinfectant, or abrasive detergents.

Hinges and corresponding screws should be replaced every 5 years. Check periodically to ensure product is in good working condition.

WARRANTY

1 Year


INSTRUCTIONS FOR USE

1. Select the appropriate indexing bar for the CT Couch Top.
2. Position the indexing bar on the treatment table.
3. Place the Baseplate or Overlay on top of the indexing bar.

ITEM INFORMATION

ITEM ID	PRODUCT DESCRIPTION	COMPATIBILITY
RNE-100AL	Turn-Lock Aluminum Indexing Bar	Exact™ Couch Top
RNE-102AL	Press-Lock Aluminum Indexing Bar	Exact™ Couch Top
RNE-103AL	Insert-Lock Aluminum Indexing Bar	iBEAM®, Novalis, and TomoTherapy® Couch Tops
RNE-104AL	C-Lock Aluminum Indexing Bar	Civco GE CT Couch Top

To be used by a qualified medical professional
IFU-A-016

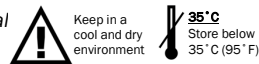

EC REP
Lotus NL B.V.
Koningin Julianaplein 10, le Verd, 2595AA
The Hague, Netherlands
T: +31645171879 (English)
T: +31626669008 (Dutch)

Klarity Medical Products, LLC
600 Industrial Parkway
Heath, OH 43056
T: 740.788.8107 F: 740-788-8109
info@klaritymedical.com
www.klaritymedical.com