

Klarity Cleanable Straps

INSTRUCTIONS FOR USE

GENERAL INFORMATION

Klarity Cleanable Straps are designed for use with patients undergoing radiation therapy.

PRODUCT DESCRIPTION

Klarity straps are created for added support, immobilization, and in some cases, a hands-free aid for Klarity Cushion™ set ups. For more information on the use of R565-605C, see IFU-C-003. Klarity Cleanable Straps use aggressive hook Velcro to secure the patient as needed during treatment.

Klarity Straps are available in a variety of sizes. Please consult our catalog or www.klaritymedical.com for a complete selection.

CLEANING AND STORAGE

Klarity Straps must be stored in a dry, clean place at a temperature between 50-95 ° F (10–35 ° C). Avoid direct sunlight.

Regular cleaning and maintenance are required to remove dirt, dust, and grime. Clean affected areas with mild soap and water, then rinse with fresh water and dry with a clean cloth. For best results treat all stains immediately. Additionally, the Klarity strap fabrics can resist specific cleaners. Use only recommended disinfectants as listed.

Recommended Disinfectants	
Accel Prevention Wipes	Diluted 1 Accel Five TB
Accel Prevention Concentrate	Diluted 1 Virox 5
Accel Fire TB	Diluted 1 Oxiver Plus
Bleach	Diluted 1 Oxiver TB Plus
Clorox Disinfecting Wipes	Diluted 1 Accel Concentrate
Diluted 1 Isopropyl Alcohol	Diluted 1 Oasis 146
Isopropyl Alcohol	Hydrogen Peroxide
Oasis 146 Multi-quat Concentrate	Oxivir plus Concentrate
Oxivir TB	Oxivir TB Wipes
Virox 5-Concentrated	

To be used by a qualified medical professional
IFU-A-025



INSTRUCTIONS FOR USE

General use:

1. Determine the desired area where immobilization is needed.
2. Wrap the strap around the area and secure the Velcro ends together.
3. Make any desired adjustments as needed for proper positioning.
4. Clean straps after use.

This strap may also be used with the Klarity Shell Kit system as R565-605C Cleanable Strap. Please refer to IFU-C-003 for instructions.

See www.klaritymedical.com/resources for a complete list of our IFU's.



AJW Technology Consulting GmbH
Konigsallee 106
40215 Dusseldorf, Germany
T: +49 211 30132232
www.ajwtech.com

Klarity Medical Products, LLC
600 Industrial Parkway
Heath, OH 43056
T: (740) 788-8107
F: (740) 788-8109
info@klaritymedical.com
www.klaritymedical.com