



KEL-C100

Stainless Steel Airflow™ Oven Cart



Assembly Instructions:

1. Lay the top shelf face down inside cardboard box.
2. Verify that the set screw is loose so that the cart legs full seat into the tube. Insert cart legs and tighten set screw.



3. Mark cart legs for the desired shelf height position.



4. Slide shelf over cart legs to desired position and tighten set screws.



5. If installing casters, remove adjustable plastic feet (if applicable). Slightly tighten nut on caster



6. Insert casters and tighten.

